MAGIC CARE®

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Healthcare at Home

Minipulsar

Sfigmomanometro da polso

Wrist blood pressure monitor

- I** MANUALE ISTRUZIONI D'USO
- GB** INSTRUCTIONS FOR USE MANUAL

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Minipulsar Wrist blood pressure monitor

**Congratulations on your purchase and thank you for choosing our unit.
Carefully read these instructions and keep them in a safe place for future reference.**

Use the unit only as described in this instruction manual.

For specific information on your own blood pressure, contact your physician.

This Blood Pressure Monitor bears the CE mark "CE 0051" and is manufactured in compliance with the essential requirements of Directive 93/42/EEC concerning medical devices, as well as the EMC directive 89/336/EEC:

- UNI EN 1060-1 (Dec. 1995) Non-invasive sphygmomanometers General requirements
- EN 1060-3 (Feb. 1997) Non-invasive sphygmomanometers Supplementary requirements for electro-mechanical blood pressure measuring systems

Blood pressure measurements determined with this blood pressure monitor are equivalent to those obtained by a trained observer using cuff/stethoscope auscultation method, within the limits prescribed by the American National Standard, Electronic or Automated Sphygmomanometers.

Disposal: In conformity with Directive 2002/96/EC on waste electrical and electronic equipment (WEEE), at the end of its life the unit should be taken to an electrical appliance collection point for its environmentally friendly disposal.

1. How the unit works

This Blood Pressure Monitor uses oscillometric method to detect your blood pressure. Before the cuff starts inflating, the device will establish a baseline cuff pressure, which is equivalent to the air pressure. The measurement of your blood pressure is based on this baseline pressure. After the cuff inflates to block your blood in the artery, the deflation process starts. During the deflation of the wrist cuff, the monitor is detecting the pressure oscillations generated by the beat-to-beat pulsatile. Any muscle movement during this period of time will cause measurement error. After detecting the amplitude and the slope of the pressure oscillations during the deflation process, the unit will determine the systolic (highest) and diastolic (lowest) pressures for you, and your pulse rate is detected at the same time.

2. About Blood Pressure

What is Blood Pressure?

Blood pressure is the pressure exerted on the artery tube while blood flows through the arteries. The pressure measured when the heart contracts and sends blood out of the heart is systolic (highest) blood pressure. The pressure measured when the heart dilates with blood flowing back into the heart is called diastolic (lowest) blood pressure.


Why Measure Your Blood Pressure?

Among the various health problems afflicting modern people, problems associated with high blood pressure are by far the most common. High blood pressure's dangerously strong correlation with cardiovascular diseases and high morbidity has made measuring blood pressures a necessity of identifying those at risk.

3. About Blood Pressure

The **World Health Organization (WHO)** and **National High Blood Pressure Education Program Coordinating Committee** have developed a blood pressure standard, according to which areas of low- and high-risk blood pressure are identified. This standard, however, is a general guideline as individual's blood pressure varies among different people and different age groups...etc.


It is important that you consult with your physician regularly. Your physician will tell you your normal blood pressure range as well as the point at which you will be considered at risk.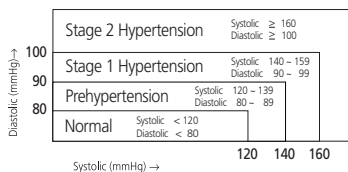

4. Blood Pressure Fluctuation

Blood pressure fluctuates all the time!


You should not be over worried if you encountered two or three measurements at high levels. Blood pressure changes over the month and even throughout the day. It is also influenced by season and temperature.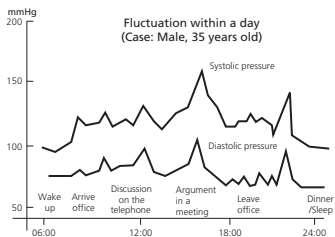


5. Carrying Case

Carrying Case:

Please store the blood pressure monitor in the carrying case as shown below: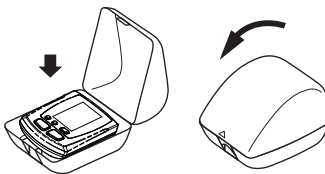

6. Name/Function of Each Part

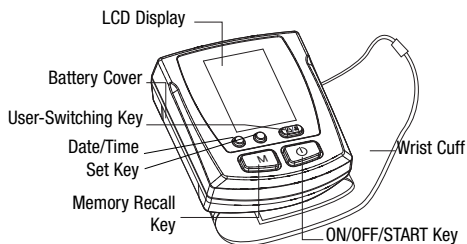
Accessories for unit:

2 "AAA" size, 1.5V alkaline batteries included in the carrying case with the unit.

7. Display Explanations

Display: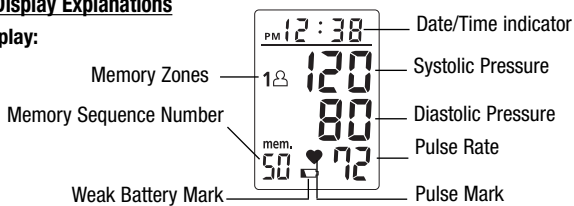

Icons:

1 **Memory Zone 1:** Appears when the monitor is operated in memory zone 1

2 **Memory Zone 2:** Appears when the monitor is operated in memory zone 2

Weak Battery Mark: Appears when batteries should be replaced

Heart Icon: Shows the pulse rate per minute

mem. **Memory Mark:** Shows the number of stored measurements

EE **Measurement Error:** Exceeding measurement range. Measure again quietly. Wrap the cuff correctly and keep wrist steady during measurement. If error keeps occurring, return the device to your dealer or an authorised service centre.


E 1 **Air Circuit Abnormality:** Measure again quietly. If error keeps occurring, return the device to your dealer or an authorised service centre.

E 2 **Pressure Exceeding 300 mmHg:** Switch the unit off and measure again quietly. If error keeps occurring, return the device to your dealer or an authorised service centre.

E 3 **Data Error:** Remove and reload the batteries. If error keeps occurring, return the device to your dealer or an authorised service centre.

8. Installing Batteries

1. Place your thumb on the side of the battery cover. Pull the battery cover in the direction of the cuff to open. Replace the batteries in pairs.
2. Install lower battery first in the bottom position with the battery ribbon underneath.
Battery Type: 2 Alkaline LR03 (AAA) size
3. Place in the other battery and pull up the battery ribbon over it.
4. Replace the cover by placing the end tabs of the battery cover in the slots on the top of the battery compartment. Click in the other end to secure the battery cover.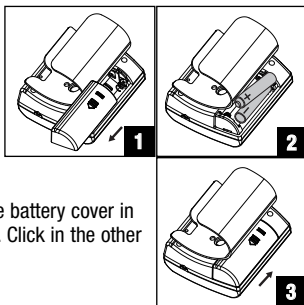

Replace the batteries if:

1. The weak battery mark appears in the display.
2. Nothing appears in the display when the power is switched on.


Remove batteries when unit is not in use for extended periods of time.

Batteries disposal: to safeguard the environment, when the batteries have reached the end of life, dispose them to an appropriate collection site.

9. Applying the Cuff


1. Remove all watches, jewelry, etc. prior to attaching the wrist monitor.
Clothing sleeves should be rolled up and the cuff should be wrapped on bare skin for correct measurements.
2. Apply cuff to left wrist with palm facing up as Fig. A.
3. Make sure the edge of the cuff is about 1 cm from the palm as Fig. B.
4. In order to insure accurate measurements, **fasten the velcro strap securely** around your wrist so there is no extra space between the cuff and the wrist as Fig. C.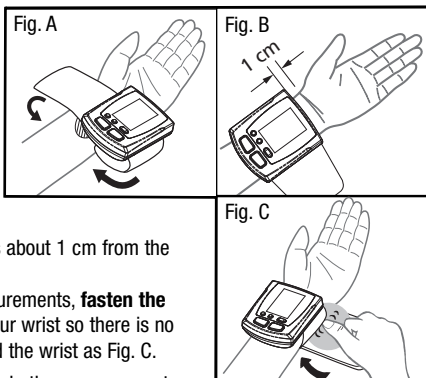

If the cuff is not wrapped tight enough, the measurement values will be false.

9. Applying the Cuff

5. If your physician has diagnosed you with poor circulation on your left arm, carefully place the cuff around your right wrist as shown in Fig. D.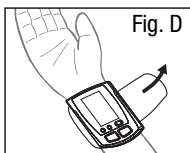


10. Correct Measuring Posture


1. Place your elbow on a table so that the cuff is at the same level as your heart as Fig. E.

Note: Your heart is located slightly below your armpit.

Relax your entire body, especially the area between your elbow and fingers.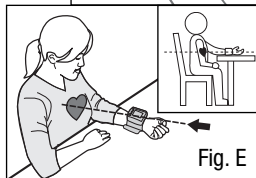


2. If the cuff is not at the same level as your heart or if you can not keep your arm completely still throughout the reading, use a soft object such as a **folded towel** to support your arm as Fig. F. Do not allow hard objects to come in contact with the wrist cuff.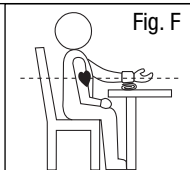

3. Turn your palm upwards.

4. Sit upright in a chair, and take 5-6 deep breaths.

Avoid leaning back while the measurement is being taken as Fig. G.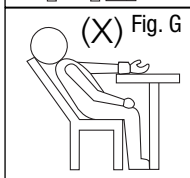


11. Helpful Tips

Here are a few helpful tips to help you obtain more accurate readings:


- Do not measure your blood pressure immediately after consuming a large meal. To obtain more accurate readings, please wait one hour before measuring.
- Do not smoke or drink alcohol before measuring your blood pressure.
- You should not be physically tired or exhausted while taking a measurement.
- It is important that you relax during measurement. Try to take 15-minute rest before a reading.
- Do not take measurements if you are under stress or under tension.
- Take your blood pressure at normal body temperature. If you are feeling cold or hot, wait a while before taking a measurement.
- If the monitor is stored at very low temperature (near freezing), have it placed at a warm location for at least one hour before using it.
- Wait about 5 minutes before taking the next pressure measurement.

12. Measurement Procedures


1. Press the **User-Switching key** to select Memory Zone 1 or Memory Zone 2.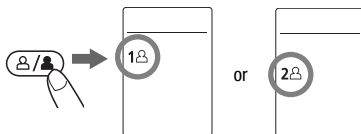

2. Place the cuff on the wrist. Press the **Power key**. All digits will light up, checking the display functions. The checking procedure will be completed after about 2 seconds.

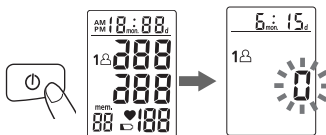
3. After all symbols appear, the display will show a blinking "0". The monitor is **Ready to Measure** and will **automatically** inflate the cuff to start measurement.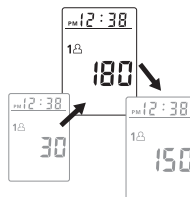

4. When the measurement is completed, systolic pressure, diastolic pressure and pulse will be shown simultaneously on the LCD screen. **The measurement is then automatically stored into the pre-designated memory zone.**

This monitor will re-inflate automatically to **approximately 220 mmHg** if the system detects that your body needs more pressure to measure your blood pressure.

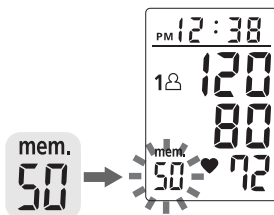
Note:

1. This monitor automatically switches off approximately **1 minute** after last key operation.
2. To interrupt the measurement, simply press any key. The cuff will deflate immediately after a key is pressed.
3. During the measurement, do not talk or move your arm or hand muscles.

13. Recalling Values from Memory

1. The Monitor has two Memory Zones (1 and 2). **Each zone can store up to 50 measurements.**
2. To read memory values from a selected Memory Zone, use the “**User-Switching Key**” to select a Memory Zone (1 or 2) from which you want to recall values from. Then, press the “**M**” key, and the values will be shown on the LCD display with an assigned memory sequence number.
3. **The monitor will display an average of the previous measurements first.** Every new press of the “**M**” key calls for one set of prior values stored. Latest memory will be recalled first. **The calculated average will be applied to a maximum of the last 3 memories.**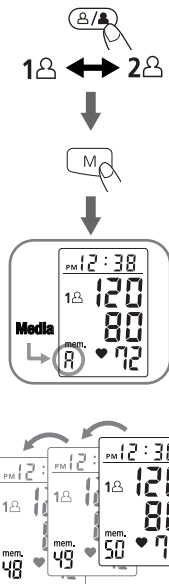


14. Clearing Values from Memory

1. Press the **User-Switching key** to select the memory zone 1 or memory zone 2.
2. Continue to press the “**M**” key for approximately 5 seconds, then the data in the pre-designated memory zone can be erased automatically.

Note: The data in both memory zones (zone 1 and zone 2) can be erased if any of the batteries is moved away.


Suggest the users to first record the data on the Blood Pressure Log (See page 21, 22) before replace or move away the batteries.

Press “**M**” key for 5 seconds to clear all memories.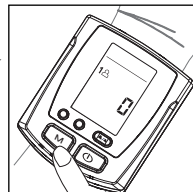

15. Time Adjustment

- To adjust the date/ time in the monitor, press the **Ⓞ Key**.
The display will show a blinking number showing the month.
- Change the month by pressing the **+ Key**.
Each press will increase the number by one in a cycling manner. Press the **Ⓞ Key** again to confirm the entry, and the screen will show a blinking number representing the date.
- Change the date, the hour, and the minute as described in Step 2 above, using the **+ Key** to change and the **Ⓞ Key** to confirm the entries.
- A **"0"** will reappear as the Blood Pressure Monitor is ready for measurement again.

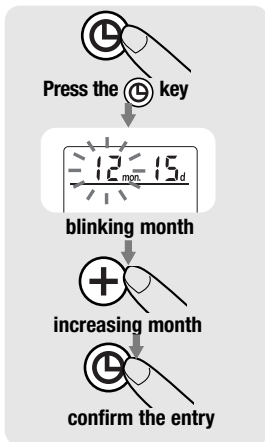16. Troubleshooting

If any abnormality will arise during use, please check the following points.

Symptoms	Check Points	Correction
No display when the Ⓞ Key is pressed	Have the batteries run down?	Replace them with two new alkaline batteries.
	Have the batteries' polarities been positioned incorrectly?	Re-insert the batteries in the correct positions.
EE mark shown on display or the blood pressure value is displayed excessively low (high)	Is the cuff placed correctly?	Wrap the cuff properly so that it is positioned correctly.
	Did you talk or move during measurement?	Measure again quietly. Keep wrist steady during measurement.
	Did you shake the wrist with the cuff on?	

Note: If the unit still does not work, return it to your dealer or an authorised service centre.
Under no circumstance should you disassemble and repair the unit by yourself.

To ensure accurate measurements, it is recommended that the performance should be checked every 2 years.

17. Cautionary Notes

- The unit contains high-precision assemblies. Therefore, avoid extreme temperatures, humidity, and direct sunlight. Avoid dropping or strongly shocking the main unit, and protect it from dust.
- Leaky batteries can damage the unit. Remove the batteries when the unit is not used for a long time.
- Do not press the  key when the cuff is not placed around the wrist.
- The unit should not be operated by children so to avoid hazardous situations.
- Do not disassemble the main unit or cuff.
- If the unit is stored near freezing, allow it to acclimate at room temperature before use.
- Repairs should be performed by authorised personnel only. Unauthorised repairs will invalidate the warranty and may represent a hazard to the user. You should not use any tool to open the device nor should you attempt to adjust anything inside the device. If you have any problems, please contact your dealer or an authorised service centre.
- For users diagnosed with common arrhythmia (atrial or ventricular premature beats or atrial fibrillation), diabetes, poor circulation of blood, kidney problems, or for users suffered from stroke, or for unconscious users, the device might not be suitable for use.
- Clean the blood pressure monitor body and the cuff carefully with a slightly damp, soft cloth. Do not press. Do not bend the pre-formed cuff inside out. Do not wash the cuff or use chemical cleaner on it. Never use thinner, alcohol or petrol (gasoline) as cleaner.
- To stop operation at any time, press the  key, and the air in the cuff will be rapidly exhausted.
- Once the inflation reaches 300 mmHg, the unit will start deflating rapidly for safety reason.

18. Specifications

Mod.:	: D400
Power source	: 3V DC Two LR03 (AAA) Alkaline batteries
Measurement method	: Oscillometric
Measurement zone	: Pressure: 40~250 mmHg Pulse: 40~199 beats/minute
Accuracy	: Pressure: ± 3 mmHg Pulse: within $\pm 5\%$ of reading
Pressure sensor	: Semi-conductor
Inflation	: Pump driven
Deflation	: Automatic Pressure release valve
Memory capacity	: 100 (50 memories for each zone x 2 zones)
Auto-shut-off	: 1 minute after last key operation
Operating conditions:	Temperature: min 10°C; max 40°C Air humidity: min 40%; max 85%
Storage conditions:	 Temperature: min -10°C; max 60°C RH Air humidity: min 10%; max 95%
Weight (without batteries)	: 120g
Wrist circumference	: 13.5~22 cm
Limited users	: Over 18 years old
Dimensions	: 82(L) x 69(W) x 66(H) mm

: Type BF

Important: check the operating instructions.

Device and cuff are designed to provide special protection against electrical shocks.

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CE
0051

Made in China

The Manufacturer reserves the right to make technical and functional changes to this product without prior notice.

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