MAGIC CARE®

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NASAL WASH

Nasal wash for the whole family

We are pleased you have purchased our product and we thank you for your trust in us. Please note that the full range of Flaem products is visible on the website www.flaemnuova.it

Read these instructions carefully before using the nasal wash. Furthermore, we recommend that you keep this manual for future consultation.

ENG

What is it?

NASAL WASH has been specifically designed for the daily cleansing of the nasal cavity for adults and children, both with saline solution or spring water. The nasal wash is portable and does not need to be plugged in to be used.

How does it work?

NASAL WASH cleanses the nasal cavity, collecting the rinsing solution in a dedicated compartment in the nasal wash bottle. It can be used with saline solutions and spring water. The ergonomically designed spray compressor body provides a comfortable grip and a button that's easy to use. There's also an air intake that reduces the noise it makes. The upper part of the body houses two adjacent chambers, one for the wash solution and the other to collect the used rinse liquid. Three different sizes of nasal adapters are available to suit the size of the nostrils, which are inserted into the top of the bottle.

When do I use it?

The **NASAL WASH** is used to clean the nasal cavity, and is useful for adults and children for daily cleansing.

ATTENTION:

The upper body, available as an accessory (code ACO598P), allows complete customisation of the system to suit every member of the family. For more hygienic use we recommend a personal adapter for each user. Use only with original FLAEM accessories.

As standard, the device includes (fig. A):

- (1) Nasal adaptor "A" with soft rubber tip, designed for children up to 3 years
Nasal adaptor "B" with soft rubber tip, designed for children from 3 to 10 years
Nasal adaptor "C" with soft rubber tip, for anyone over the age of 10
- (2) Separator
- (3) Spray nozzle
- (4) Extractable and washable upper body comprising 2 separate chambers:
 - (4a) Distribution chamber of 15 ml
 - (4b) Collection chamber
- (5) Wash compressor chamber with ergonomic grip
 - (5a) Dispense button
 - (5a) Battery compartment
 - (5c) Suction valve with cover

1. Inserting the batteries

1. Open the battery compartment (5b) by pressing and lifting the cover in the direction of the arrows (fig. B).
2. Insert 2 batteries into the battery compartment, ensuring they correspond with the polarities indicated in the battery compartment. Use 2 alkaline batteries (AA).
3. Replace the cover over the battery compartment by closing the top end of the hatch until it clips shut.

Remove the batteries if the device will not be used for a long time. Batteries left in the battery compartment for long periods may leak and could damage the device.

ATTENTION:

- The device does not contain any parts that can be repaired by the user. The warranty does not cover the batteries or damage caused by old batteries.
- Use only batteries of known brands. Always replace all old batteries with new ones. Do not mix old and new batteries. Use batteries of the same brand and same type.

Battery disposal: Spent batteries must be disposed of in appropriate containers.

2. Instructions for use

- **Before using, wash you hands and clean the nasal wash and accessories as shown in paragraph 3. Cleaning, sanitising and disinfecting.**

- **During application, take care as the solution can drip from the nose.**

- 2.1 Before use, insert the batteries as shown in paragraph 1.
- 2.2 To use the nasal wash correctly, the man icon (♂) (4c), on the upper part of the bottle (4) should be facing the user. The upper part of the bottle can be attached with the man icon on the same side as the button (5a) enabling the user to activate the nasal spray with the thumb, or with the icon on the side opposite to the button, enabling the user to activate the nasal spray with the index finger. When attaching the upper section (4) to the body of the nasal wash compressor body (5) (fig. C), start with the man icon opposite to the side you want it to end up on then turn clockwise until it reaches the desired position (fig. C).
- 2.3 Pour the solution (max 15 ml) into the distribution chamber (4a) of the upper section, as shown in (fig. E) Check that the nozzle is in place (3).
- 2.4 Apply the separator (2) aligning it so that the ball (2a) is in line with the man icon (♂) (4c) located on the outside of the upper section of the bottle (4), as shown in figure (fig. F). Next, insert the nasal adapter most suited to the size of your nostrils (1A-1B-1C).
- 2.5 Insert the end of the nasal adapter into one nostril (fig. G).
- 2.6 Activate the nasal wash by holding down the button (5a) while inhaling through the nose and release the button before exhaling; when exhaling, blow gently through the nostrils to help expel the mucus, while holding the other nostril closed.
- 2.7 It is also better to hold the other nostril closed with a finger while sucking up the nasal wash.
- 2.8 Repeat the operation from point **2.5** to point **2.7** for the other nostril, trying to distribute the solution evenly between the two nostrils.
- 2.9 For children or anyone unable to correctly coordinate the steps in point **2.6** without assistance, we recommend alternating the application between the nostrils every 15 seconds, leaving suitable pauses to allow them to expel the mucus.
- 2.10 After application, turn the device off and clean and/or disinfect it following the instructions provided in paragraph 3. Cleaning and disinfecting.

3. Cleaning, sanitising and disinfecting.

The device must be cleaned thoroughly to ensure that it works correctly and performs reliably. After use, take the nasal wash apart as shown in the diagram (see fig. A), remove the batteries and clean and/or sanitise the parts as follows:

3.1 Cleaning the nasal wash compressor body, suction valve

Use only a damp cloth with antibacterial soap (non-abrasive and with no solvents of any sort). The nasal wash has a suction valve (5c) that needs to be cleaned when dirty. To access the valve, lift the batter compartment cover and remove the valve using a pen or similar object as a lever (fig. H); wash the valve with soap and water. Ensure the valve is completely dry before reassembling.

ATTENTION:

The nasal spray compressor body is not waterproof, therefore do not immerse the body in water or wash under running water.

3.2 Sanitisation: Sanitise the parts (1A-1B-1C,2,3,4) using one of the following methods.

Method A: under warm drinking water (approx. 40°C) with delicate (non abrasive) washing up liquid.

Method B: immerse in a solution of 50% water and 50% white vinegar. Finally, rinse thoroughly

with warm drinking water (approx. 40°C);

Method C: boil in water for 20 minutes; it is preferable to use de-mineralised or distilled water to prevent calcium deposits.

Drying: After having disinfected the accessories, shake them vigorously and place them on a paper towel or dry them with a jet of hot air (e.g. with a hair dryer).

3.3 Disinfection: Accessories (1A-1B-1C,2,3,4) can be disinfected

The disinfection procedure described in this paragraph is to be followed before and after using the accessories. It is effective on the parts that undergo this treatment only if each step is followed and only if the parts to be treated have previously been sanitised as described in the previous SANITISING section. The disinfectant must be an electrolytic chloroxidizer (active ingredient: sodium hypochlorite) specifically for disinfecting.

Implementation:

- Fill a container big enough to hold all of the parts to disinfect with a solution of potable water and disinfectant, according to the proportions indicated on the packaging of the disinfectant.
- Completely immerse each part in the solution, taking care to avoid the formation of air bubbles on the parts. Leave the parts immersed for the amount of time indicated on the packaging of the disinfectant associated with the concentration chosen for the solution.
- Remove the disinfected parts and rinse abundantly with warm potable water.
- After having disinfected the accessories, shake them vigorously and place them on a paper towel. Alternatively, dry them with a jet of hot air (for example, a hair dryer).
- Dispose of the solution following the instructions provided by the disinfectant manufacturer.

4. Important warnings

- Do not wash the device under running water or by immersion and keep it safe from being sprayed by water or other liquids.
- Do not use the nasal wash while in the bath or shower.
- Do not expose the device and the batteries to extreme temperatures. Keep the device away from sources of heat, direct sunlight or excessively warm locations.

As with any electrical device, especially in the presence of children, **NASAL WASH** requires certain precautions.

- The device may be used by children not younger than 8 years old and people with limited physical, sensory or mental capacity, without experience or necessary information, if under supervision, or after receiving the instructions on how to use the device safely and having understood the inherent risks. Children must not play with the device. The cleaning and maintenance required of the user must not be carried out by children without supervision.
- Never use batteries different to those indicated on the technical specifications plate.
- Remove the batteries before putting away the **NASAL WASH**.
- The casing of the device is not waterproof.

- Repairs must be carried out only by authorised personnel. Unauthorised repairs void the warranty and may be hazardous for the user.
- **ATTENTION:** Do not modify this device without authorisation from the manufacturer.
- The Manufacturer, Importer and Seller will be held responsible for the safety, reliability and performance, only if the device is used in accordance with the instruction manual.
- **Microbial contamination:** we do not recommended sharing the nasal wash and accessories.
- **Interactions:** the materials used have been tested in accordance with bio-compatibility regulations (ISO 10993-5 and ISO 10993-10).
- Clean the device after each use, to ensure optimum hygiene and better function and durability of the device.
- Do not obstruct the suction valve on the bottom of the spray compressor body; never insert anything into the cover.

This text is for users in countries except European countries:

- This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety.
- Children should be supervised to ensure that they do not play with the appliance.

SYMBOLS

Batteries ()

In accordance with directive 2006/95 CE and subsequent modifications

Read the instruction manual

IP45

Protected against solid foreign bodies of \varnothing 1 mm or greater and jets of water

Phthalates and BPA free

5. Fault identification

PROBLEMS	SOLUTIONS
5.1 - If the nasal wash dispenses too little or too briefly.	The batteries may be running out. Change the batteries. Check that the spray nozzle is inserted correctly (3). Check that the spray nozzle is clean and, if not, clean the nozzle, following the instructions provided in the manual carefully regarding Cleaning, sanitising and disinfecting .
5.2 - If the nasal wash does not dispense the solution	The batteries may be running out or incorrectly inserted. Change the batteries. Check that the solution has been correctly poured into the dispensing chamber(4a) of the upper body (4). Check that the quantity of solution does not exceed 15 ml. Check that the spray nozzle is clean and, if not, clean the nozzle, following the instructions provided in the manual carefully regarding Cleaning, sanitising and disinfecting .

NOTE: If, after checking the aforementioned conditions, the appliance still does not distribute the wash, we recommend that you contact your dealer or an authorised Flaem service centre.

6. Technical specifications

Model: Nasal Wash

Power supply voltage	3.0 V  4.5 W
Max dispensing chamber capacity	15 ml
Dispensing time for 10 ml of saline solution or spring water, approximately 2 minutes (1 ml in around 12 seconds) Oversize Cumulative Percentage in mass > 10 micron between 85 and 96 %	
Nasal wash size	7x6x15.5h cm
Nasal wash weight (without batteries)	185 g
Marking	
Nasal wash batteries	2 alkaline batteries 1.5 V (AA)
Operating Conditions:	Temperature min 10°C; max 40°C Air humidity: min 10%; max 75% RH
Storage conditions:	Temperature: min -10°C; max +35°C Air humidity: min 10%; max 75% RH
Operating/storage atmospheric pressure:	min. 690 hPa; max. 1060 hPa P

 **Disposal:** Bear in mind that the product contains batteries and dispose of correctly. In compliance with the Directive 2002/96/CE, the symbol printed on the device shows that the device to be  disposed of is considered waste and must therefore be an item of "differentiated collection". Consequently, the user must take it (or have it taken) to the differentiated collection sites provided by the local authorities, or turn it in to the dealer when purchasing an equivalent new device. Differentiated waste collection and the subsequent treatment, recycling and disposal procedures promote the production of devices made with recycled materials and limit the negative effects on the environment and on health caused by potential improper waste management. Unauthorised disposal of the product by the user shall result in the application of administrative penalties, as per article 50 and the subsequent revisions of Italian Legislative Decree no. 22/1997.

Electromagnetic compatibility

This device was designed to satisfy the current requirements for electromagnetic compatibility. The manufacturer reserves the right to make technical and functional modifications to the product with no prior warning.

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© Copyright by FLAEM NUOVA S.p.A.,
FLAEM NUOVA S.p.A.