# MAGIC CARE®

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Ultrasonic humidifier.

**Principle of operation**

The ultrasound humidifier uses high frequency oscillations that cannot be heard by the human ear to break the water down into tiny particles with a diameter of approximately 1 – 5 µm. The built-in fan system propels this mist of particles into the air. This causes an increase in the humidity of the surrounding environment.

**Main features**

- Easy-to-fill tank
- Automatic shutoff when water top-up required
- Cool fog
- Vapour flow adjustment
- Nightlight
- Low consumption (30 W)
- Silent operation < 35 dB (A)

**Indications**

- Increases and regulates the relative humidity of the air in the surrounding environment.
- Eliminates electrostatic charges.

**Main components of the unit**

- |   |   |
|---|---|
| <ul style="list-style-type: none"> <li>1 - Tank</li> <li>2 - Vapour outlet (cold)</li> <li>3 - Top-up lid</li> <li>4 - Ultrasound chamber</li> <li>5 - ON/OFF and adjustment knob</li> <li>6 - Nightlight switch</li> </ul> | <ul style="list-style-type: none"> <li>7 - Operation Indicator light</li> <li>- Blue light: unit on</li> <li>- Red light: more water required</li> <li>8 - Transducer</li> <li>9 - Float</li> <li>10 - Power supply cord</li> </ul> |
|---|---|

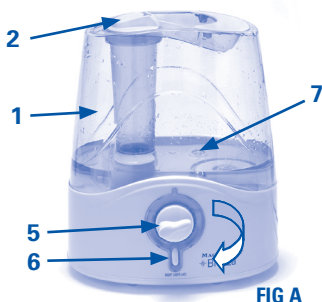
FIG A

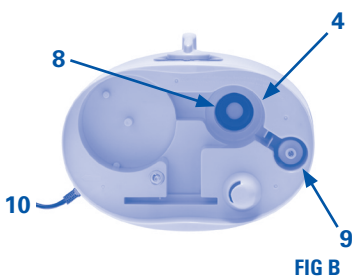
FIG B

FIG C

FIG D

**Important cautionary notes**

⚠ The use of any electrical appliance must respect a few fundamental rules:

- When opening the packaging, make sure the appliance is intact, particularly the plastic parts, which, if damaged, may expose live internal parts of the appliance. If in doubt, contact an authorised After Sales Service.
- Packaging materials must be properly disposed of and kept out of children's reach.
- Read the instructions carefully before use and follow the indications provided.
- Check that the data on the specification plate correspond to those of the electrical supply.
- Use of single or multiple adaptors is not recommended. When using an extension lead, make sure that the cord is completely uncoiled.
- Do not plug or unplug the recharger base with wet hands.
- Never pull on the lead, or on the appliance, to disconnect the plug from the socket.

- This appliance can be used by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved. Children shall not play with the appliance. Cleaning and user maintenance shall not be made by children without supervision.
- Keep the unit away from sources of heat.
- Keep the unit away from electric and/or electronic appliances.
- This unit must not be used by children or persons with physical, mental and/or sensory impairment, and those lacking experience or knowledge of the use of electric appliances.
- The unit is not a toy. Keep out of the reach of children.
- If the power supply cord is broken or worn, contact an authorised service centre for replacement.
- In the event of breakdown or malfunction, switch the appliance off immediately, disconnect from the power supply and contact your local service centre.
- Do not touch the water in the tank, or components of it whilst the appliance is switched on.
- Never switch the appliance on when the water tank is empty.
- Do not wet the unit or place it in water.
- For indoor use only. Protect it from exposure to atmospheric agents.
- This appliance must be used for its intended use only. Using it for any other purpose is both improper and dangerous. The manufacturer cannot be held responsible for any damage deriving from improper or incorrect use.
- Always use cold water (never hot). Do not add essences, perfumes, medicines or any other substance.
- Always switch off the appliance and disconnect from the power supply before removing the water tank.
- Position the humidifier on a flat, stable and waterproof surface.
- The humidifier must be kept away from curtains, rugs, furniture or any other moisture-sensitive objects.
- When not in use, disconnect from the power supply and store in a suitable place.

## Operating Instructions

- Keep the appliance at room temperature for 30 minutes before using.
- Use the appliance at a room temperature of between +5°C and +40°C.
- Remove the water tank from the unit FIG. D. Unscrew the top-up lid (3) and fill the tank with tap water\* FIG. C (never use hot water). Unscrew the top-up lid (3) and reposition the tank on the appliance.
- Insert the power supply cord (10) in the power socket.
- Turn the knob (5) clockwise to switch the appliance on FIG A.
- Turn the knob further (5) to adjust the amount of mist dispensed (2)
- The mist is not dispensed instantly, it may take a few seconds to fill the chamber (4).
- Operate the night light using the ON  /OFF  switch (6).
- When the water in the tank runs out the blue light (7) becomes red. Switch the unit off and pull out the plug

## Maintenance

When using the unit and particularly in places where the water is very hard, i.e. it contains too much calcium and magnesium, the presence of these two substances may leave a white film and a thick layer of lime scale may form on the walls of the tank and transducer, thus reducing their efficacy.

Always comply with the following precautions:

- \*Use previously boiled cold water or distilled or demineralised water.
  - Clean the transducer, chamber and water tank with a damp cloth each time it is topped up.
- Keep the components completely dry when the appliance is not in use.

## Cleaning

Make up approximately 1L of a 50-50 mixture of water and white vinegar.

- Unscrew the top-up lid (3) and fill the tank FIG. C.
  - Screw the top-up lid (3) and reposition the tank on the appliance.
- Wait for about 1 hour. (ABSOLUTELY DO NOT OPERATE THE UNIT)
- Empty both the tank and base of the humidifier.
  - Rinse the tank well with water (**NEVER RINSE THE HUMIDIFIER BASE**).
  - Dry the base of the humidifier with a cloth.

For correct appliance operation, the cleaning procedure should be performed once a week during use.

## Troubleshooting

Problem	Probable cause	Solution
The operation light fails to go on.	Power cord disconnected	Check that the power cord is connected to the socket.
	Switch off	Activate the switch by turning the knob (4) clockwise.
The operation indicator light is on but no vapour comes out.	Water tank empty	Vapour regulator at minimum.
	Fill water tank	Turn the regulation knob clockwise.
The mist has an unpleasant odour.	Using the unit for the first time	Empty the tank of water and clean following the procedure described in the CLEANING chapter
	Appliance already used	
The operation light is on but the appliance does not work	The water in the chamber is beyond the maximum level	Remove surplus water
	The tank is not positioned correctly on the chamber	Replace the tank on the chamber correctly.
No mist is visible, despite the power being at maximum	Stagnant water in the tank	Empty the tank of water and clean following the procedure described in the CLEANING chapter
	Lime scale present	
Strange noise	Resonance of water in the tank when lower than the minimum level	Fill the tank with water
	The appliance is on an unstable and/or sloping surface.	Position the unit on a flat and stable surface

## Technical details

Mod. P0509ED	
Power voltage	220 – 240 V~
Nebulisation	±220 ml / h
Frequency	50 / 60 Hz
Noise levels	< 35 dB (A)
Power	30 W
Dimensions	26 X 20 X 32 cm
Tank capacity:	5 l
Marking	
Approvals	

 Disposal: In conformity with Directive 2002/96/EC, the symbol shown on the appliance to be disposed of indicates that this is considered as waste and must therefore undergo «sorted waste collection». The user will therefore have to take the above waste (or have it taken) to pre-sorted waste collection centres organised by the local authorities, or else return it to the dealer at the time of purchasing a new appliance of the same type. The pre-sorted collection of the waste and subsequent treatment, recovery and disposal operations favour the production of appliances made from recycled materials and limit negative effects on the environment and on health caused by any incorrect waste management. The unlawful disposal of the product by the user could result in administrative fines as provided by article 50 and subsequent modifications of the Italian Legislative Decree no. 22/1997.

## SYMBOLS

-  Class II unit
-  Important: check the operating instructions
-  Alternate current
-  Switch OFF
-  Switch ON

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**FLAEM NUOVA S.p.A.**